



## Eagle Eye Anesthesia

11233 St John's Industrial Parkway S.  
Jacksonville, FL 32246  
800-760-6976  
www.eagleeyeanesthesia.com

### Bain Block Installation Instructions for the Millennium Anesthesia Machine

This instruction sheet describes how to install the Eagle Eye Bain Block to the Millennium Anesthesia. (It can also be mounted to the pole of any anesthesia machine by using a pole mounting clamp - available for either a 1-3/4" or 2 " diameter pole.)

#### Items provided by Eagle Eye Anesthesia:

Bain Block assembly  
hex-head screws (2) and 2 stainless steel washers  
3/16" hex wrench (included with your Millennium packet)



Fig. 1

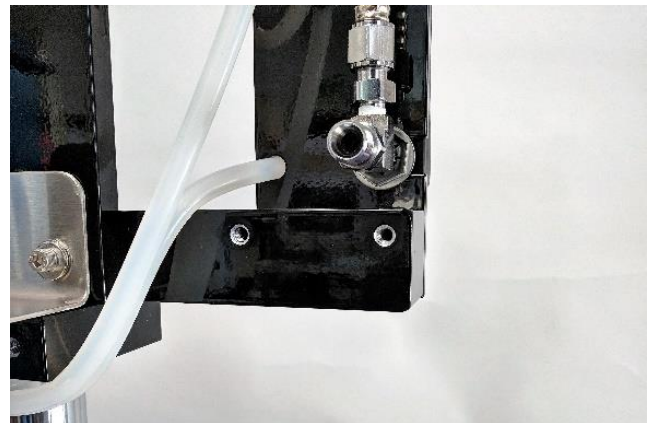


Fig. 2

#### Instructions:

- The pre-assembled Bain Block mounts to the 2 screw holes located behind the flow meter plate with supplied screws and washers (Figs 1 and 2)
- By using the supplied Allen Wrench, install the 2 screws and washers through the mounting plate and into the T-bar (cross bar) of the machine. Tighten until snug. (Fig. 3).
- The Bain breathing circuit attaches to the front nozzle. (Fig. 4)
- The bag attaches to the bag bushing underneath the block.
- The fresh gas supply tube attached to the common gas outlet after first removing the rebreathing supply tube. (Fig. 5)

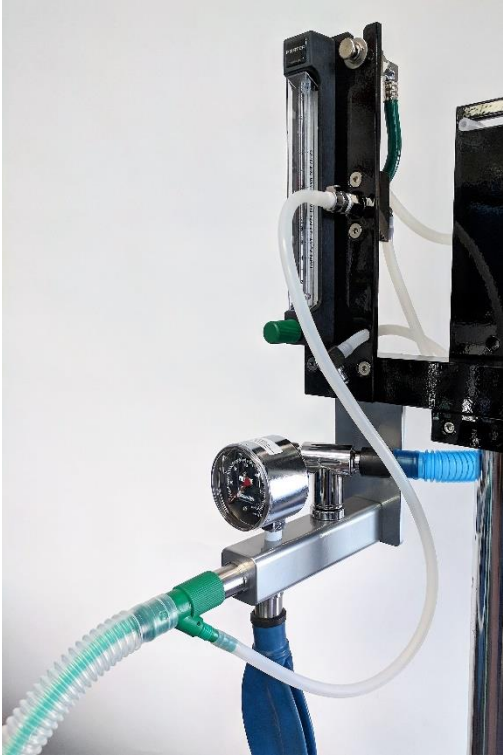


Fig. 3



## Eagle Eye Anesthesia

11233 St John's Industrial Parkway S.  
Jacksonville, FL 32246  
800-760-6976  
[www.eagleeyeanesthesia.com](http://www.eagleeyeanesthesia.com)



*Fig. 4*



*Fig. 5*

If you have any questions, give us a call: 800-760-6976.