



## Eagle Eye Anesthesia

11233 St John's Industrial Parkway S.  
Jacksonville, FL 32246  
800-760-6976  
[www.eagleeyeanesthesia.com](http://www.eagleeyeanesthesia.com)

---

### To install replacement needle valve for oxygen flow meter

#### Items needed:

- 9/16 inch wrench or adjustable wrench.
- Replacement needle valve assembly.

#### Remove old needle valve:

- Turn oxygen off at source (tank) or disconnect the machine from the oxygen supply.
- Purge O<sub>2</sub> from machine using oxygen flush valve
- Open needle valve on flow meter all the way to reveal the black flats of the needle valve assembly.
- Using a 9/16<sup>th</sup> wrench or an adjustable wrench, place on the black flats of the assembly and unscrew from the flow meter body.

#### Install replacement:

- Open replacement valve all the way
- Start threading using fingers, then tighten firmly with wrench using the black flats of the assembly ...do not over tighten.
- **\*\*\*IMPORTANT\*\*\* close needle valve prior to turning oxygen back on or connecting machine to oxygen supply.**
- (Did you close the needle valve? See above)
- Turn on oxygen at source (tank) and check for proper function.

If you have any questions, don't hesitate to call

**Eagle Eye Anesthesia**  
**904-739-4721 or 800-760-6976**